

12D MAX


PRODUCT USER MANUAL


...

Innovative technology - rejuvenate youth

LEFT

RIGHT

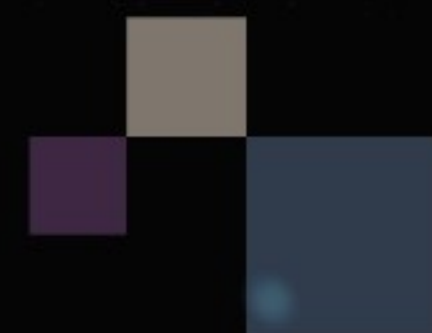

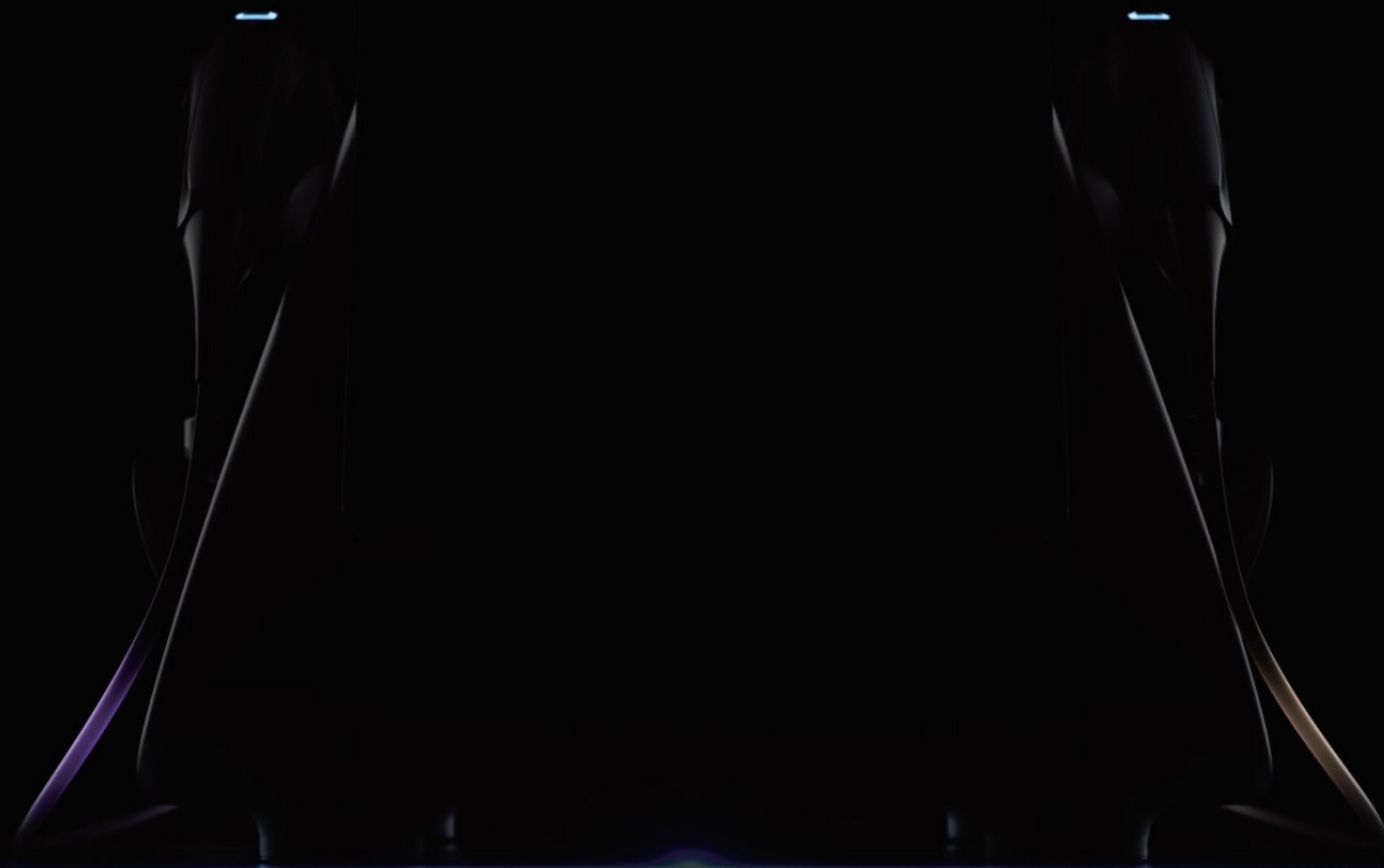

Please read the instrument manual in detail before operating,
Avoid errors during operation.

CONTENTS

12D MAX

01 Matters needing attention

- Operational taboo
- Precautions before operation
- Safety precautions
- Operation precautions
- Precautions after operation

02 Instrument installation

- Instructions for use
- Instrument assembly
- LEFT Ultrasonic handle
- RIGHT Ultrasonic handle
- Superpulse ultrasound

03 Matters needing attention

- Interface of the handle probe is described
- This section describes operation parameters
- This section describes the storage mode page
- Operate the background Settings screen

04 Configuration parameters and maintenance matters

- Instrument maintenance
- Instrument configuration parameter
- Safety tips
- Note instructions



MATTERS NEEDING ATTENTION

- Operational taboo
- Precautions before operation
- Safety precautions
- Operation precautions
- Precautions after operation



Operational taboo

1. Children, pregnant women, breastfeeding prohibited.
2. People with irregular heart and heartbeat, especially those with cardiac pacemakers, should not do this.
3. Cancer patients, major surgery patients, diabetes patients prohibited.
4. People with epilepsy and severe hypertension are prohibited.
5. If the skin has a wound, avoid the wound or do not do.
7. Do not do it in areas with artificial fillings.
6. If there is any metal in the body, do it carefully and avoid metal parts when operating.
8. If the skin has serious infectious diseases, do it carefully.

Precautions before operation

1. Consult experts or receive professional training before use.
2. Consult the customer's skin condition, allergies (severe allergies can not be operated).
3. Ask customers about instruments they have operated recently.
4. Tell the customer in detail the feeling or reaction during the operation, so that the customer is psychologically prepared.
5. Do not take medication or drink alcohol for 24 hours before and after operation.
6. Before operation, guests are required to remove the metal accessories worn on their bodies.
7. Before operation, the parameters of the beauty head should be adjusted well.
8. Check whether the instrument is in normal condition, and ensure that the instrument is normal and the preparation is complete before operation.

Safety precautions

1. When the power plug is dirty, please clean it in time and make sure that there is no foreign matter before connecting the socket.
2. It must be connected to the power socket of 110-220V, 50Hz /60hz, and the power cord must be unplugged when moving the device.
3. Do not touch the power plug and do not pull the power cord.
4. Do not touch the power cable or power plug with wet hands.
5. Do not force the area around the instrument or power supply.
6. Do not use the instrument at high temperatures, and do not place heating equipment near the instrument.
7. Do not disassemble the power socket after the instrument is started, which will cause failure.
8. When the instrument is in use, if there is smoke or odor, please unplug the power supply and consult after-sales service.
9. Do not touch your lips or eyes when the handle is activated to avoid injury.
10. If the indicator is always bright blue during operation or an abnormal sound is heard, remove the handle from the skin surface in time.

Operation precautions

1. Please read the instruction manual carefully, and then use according to the method in the instruction manual. If you do not use the instrument for a long time, please unplug the power supply.
2. When using the handle, first lower the strength, and then gradually adjust to the appropriate strength.
3. Discomfort during operation is a normal reaction, and the handle should be close to the skin during operation to avoid strengthening the discomfort.
4. When the handle is working (the work indicator is on), the handle should not leave the skin surface, and can only leave after the pause or end.
5. If the handle is abnormal, remove the handle from your skin in time to avoid unnecessary damage.
6. When the beautician operates, do not wrap the handle cable around the hand.
7. When operating, the corresponding accessories must be obeyed to avoid uncomfortable feelings.
8. During operation, there must be sufficient medium (such as gel) for the parts to be cared for.
9. Avoid the belly button, ribs, and other bulging bones.
10. When the handle is not needed, it should be placed on the accessory rack. Do not pull the connection cable hard when the handle is in the accessory rack.
11. After using the instrument, wipe it clean with cotton cloth and cotton swabs, and then wipe the metal part of the cosmetic head/handle with 75% alcohol.
12. If you experience any abnormalities while using the instrument, stop the operation immediately and consult an expert.
13. If the instrument is abnormal, please confirm the instructions again before operation; If the exception is not lifted, do not disassemble the instrument yourself, you should contact after-sales service consultation.

Precautions after operation

1. It is not recommended to accept injection, wire burying, filling and other behaviors within one month.
2. Do not eat soy sauce and other heavily pigmented food, do not drink, smoke, eat hot pepper, seafood, blood and other food, do not eat hard food such as sugar cane, betel nut; Do not take antibiotics such as aspirin.
3. Pay attention to rest after operation, ensure enough sleep, avoid strenuous exercise, and accelerate blood circulation after exercise is not easy to recover.
4. In order to achieve a better state after operation, it is necessary to do a good job of repair after operation, nutrition and water supplement; Avoid the situation that the effect is not ideal and the duration is not long due to the neglect of care and repair after operation.
5. The part of the operation will have a feeling of acid swelling in a very small position within a month, which is a normal phenomenon.
6. Please consciously abide by the above regulations, has a decisive role in the presentation of the later effect, the more standardized the later care, the better the effect (keep longer, the skin condition will be better).



INSTRUMENT INSTALLATION

- Instructions for use
- Instrument assembly
- LEFT Ultrasonic handle
- RIGHT Ultrasonic handle
- Superpulse ultrasound



Instructions for use

● Unpacking and inspection


- Carefully take out the instrument and accessories from the packaging box, and keep the packaging materials for future transportation or storage.
- Check the accessories according to the list of accessories and check whether there is any mechanical damage.

● Instrument instructions



Installation screen

Align the card slot at the back of the screen with the instrument installation interface and push it down to the bottom.

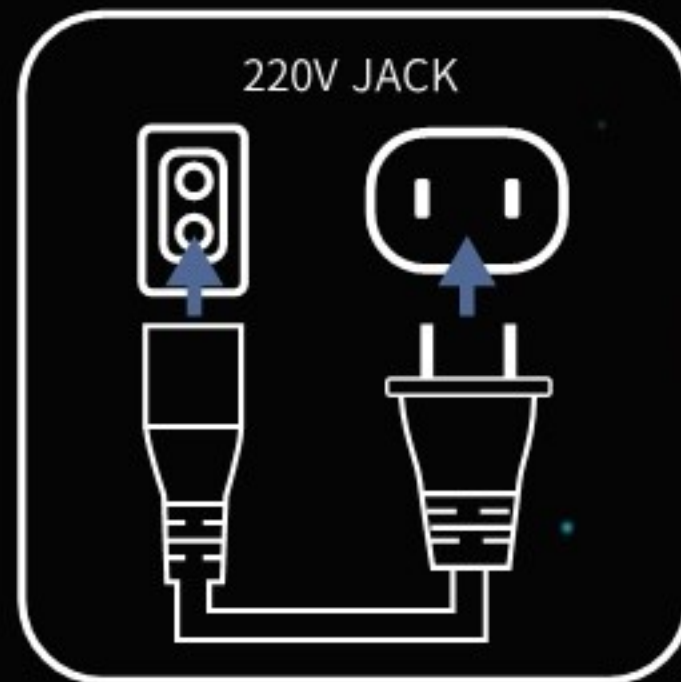

Startup operation

Connect the power cord and press the ON/OFF button on the back of the host.



connection handle

Align the handle wire plug with the interface, push it to the end, and the two handles are not connected to the interface.



Insert operating probe

According to the operation needs, the corresponding treatment probe is inserted in the direction of the hole.



Instrument assembly

13.3 Inch HD smart screen

- Patented original design
- Full sense of technology
- Full dynamic design



LEFT Ultrasonic handle

- Two-way output technique
- Large focal areas gather energy
- Multi-mode combined nursing



RIGHT Ultrasonic handle

- Two-way output technique
- Large focal areas gather energy
- Multi-mode combined nursing

LEFT Superpulse handle

-  Two-way output technique
-  MP Pattern(Multi-point line)
-  Large focal areas gather energy
-  Multi-mode combined nursing

Probe model matching

- | | |
|---|---|
|  | Face - 1.5 5.5MHz
For epidermal layer, light lines, skin tone, skin type, shrink pores. |
|  | Face - 3.0 5.5MHz
It reaches the dermis and stimulates collagen growth. |
|  | Face - 4.5 4.0MHz
Reach the fascia layer of SMAS and tighten the muscle base to achieve a firming lift. |
|  | Chin - 6.0 4.0MHz
For the skin layer, skin color, skin texture and pore shrinkage. |



pay attention to:

1. Apply gel to the skin before operation;
2. The operating handle is vertical to the skin;
3. Do not leave the skin during operation.



Operation keys
Work indicator

Handle replacement
probe

Replace the
probe key

pay attention to:

1. Apply gel to the skin before operation;
2. The operating handle is vertical to the skin;
3. Do not leave the skin during operation.

RIGHT Superpulse handle

-  Two-way output technique
-  MP Pattern(Multi-point line)
-  Large focal areas gather energy
-  Multi-mode combined nursing

Probe model matching



Eye-1.5 5.5MHz

For epidermal layer, light lines, skin tone, skin type, shrink pores.



Body - 8.0 2.0MHz

Target the epidermal layer, skin color, skin type, and shrink pores.



Body - 13.0 2.0MHz

It reaches the dermis and stimulates collagen growth.




Body - 16.0 2.0MHz

Reach the fascia layer of SMAS and tighten the muscle base to achieve a firming lift.

Superpulse ultrasound

Single mode and multi-mode combined nursing

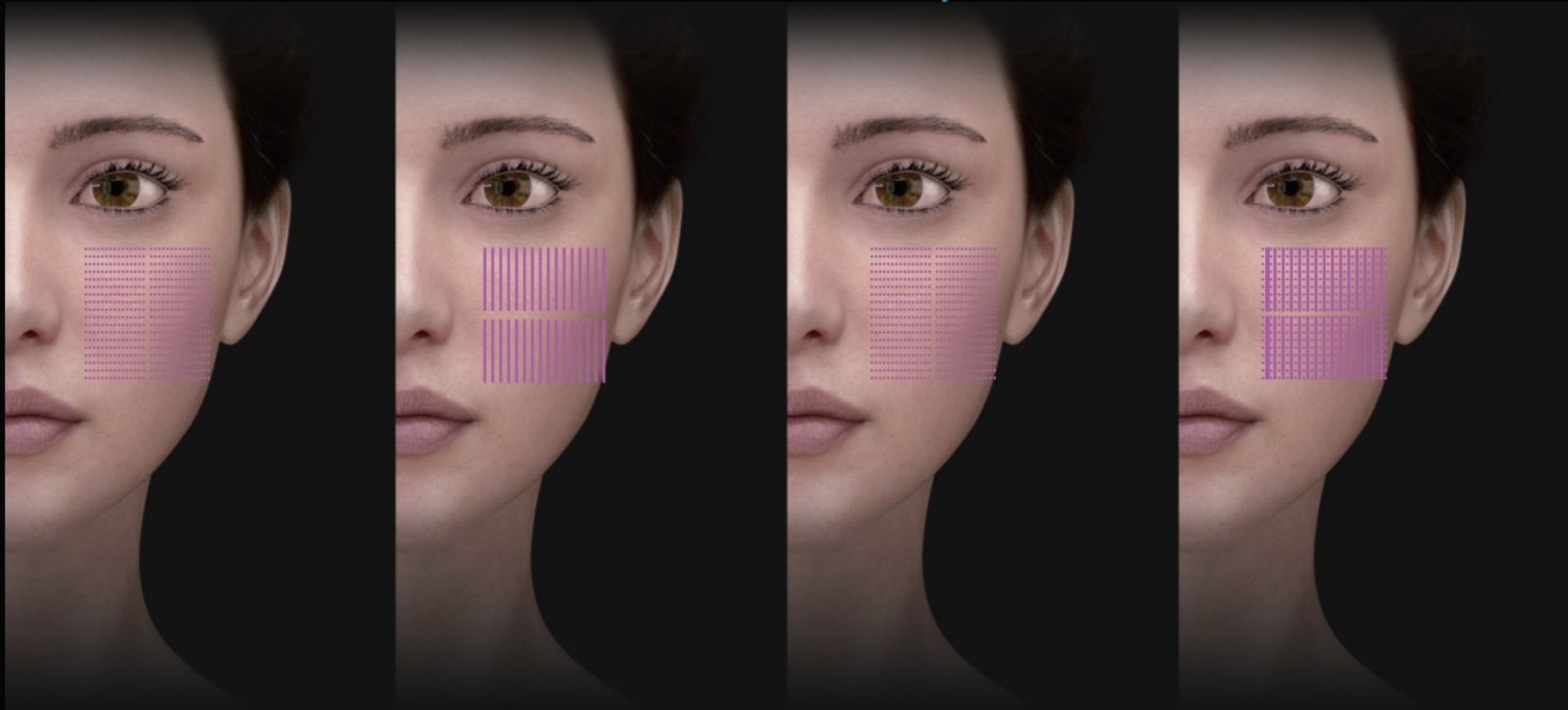

- High frequency forward and reverse linkage workDual Handle Collaborative Work.
- Multi-frequency ultrasonic treatment probe on the skin layer to the point, surface, line and skin layer !
 - Comprehensive customized solutions for operating areas and organizational levels.



MATTERS NEEDING ATTENTION


- Interface of the handle probe is described
- This section describes operation parameters
- This section describes the storage mode page
- Operate the background Settings screen



High frequency forward and reverse linkage work
Dual Handle Collaborative Work



Interface of the handle probe is described

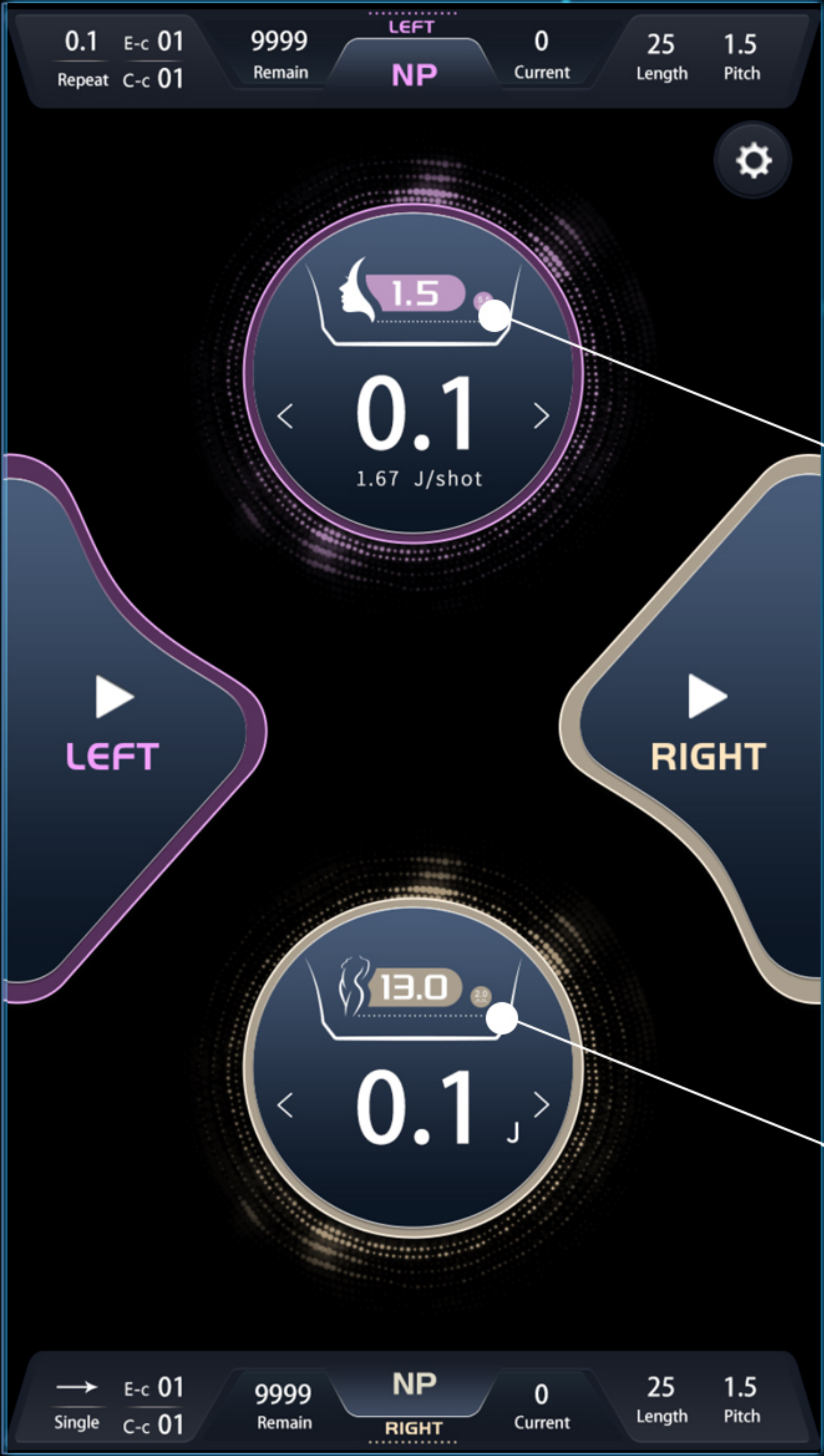

LEFT Ultrasonic probe

- Facial lift and skin firmness.
- Display the current handle with which probe handle.



Face-1.5 5.5MHz



Face-3.0 5.5MHz



Face-4.5 4.0MHz



Chin-6.0 4.0MHz

RIGHT Ultrasonic probe

- Restores body shape and skin firmness
- Display the current handle with which probe handle.



Eye-1.5 5.5MHz



Body-8.0 2.0MHz



Body-13.0 2.0MHz



Body-16.0 2.0MHz

This section describes operation parameters

Single mode and Repeat mode switch back and forth
(Single mode Unidirectional output)

Click to select the parameter modes set on the three interfaces.

The total length of the handle probe dotting,
which can be selected by clicking.

▶ LEFT handle power adjustment ready, click start.

|| After the LEFT handle probe is operated, click Pause.

RIGHT Model and frequency of handle probe.

The energy adjustment size of the handle.
(energy adjustment range: 0.1J - 3.0J)

The MP and Nomal modes are switched
(In Nomal mode, there is no J/shot symbol under the energy value)

Repeat mode two-way output, click to select parameters
(range 0.1--1.0 sec)

Single mode and Repeat mode switch back and forth



The MP and Nomal modes are switched (In MP mode, there is a J/shot identifier below the energy value)

Parameter setting

LEFT Model and frequency of handle probe.

Automatic matching according to probe and energy value (Range 1.67--50.00)

▶ RIGHT handle power adjustment ready, click start.


|| After the RIGHT handle probe is operated, click Pause.

The current total number of rounds of the handle probe / the total number of rounds remaining in use.

The interval length of the probe dots can be adjusted and selected by clicking.

Click on adjustable probe dot interval length
(optional range 1.0-2.0)

This section describes the storage mode page

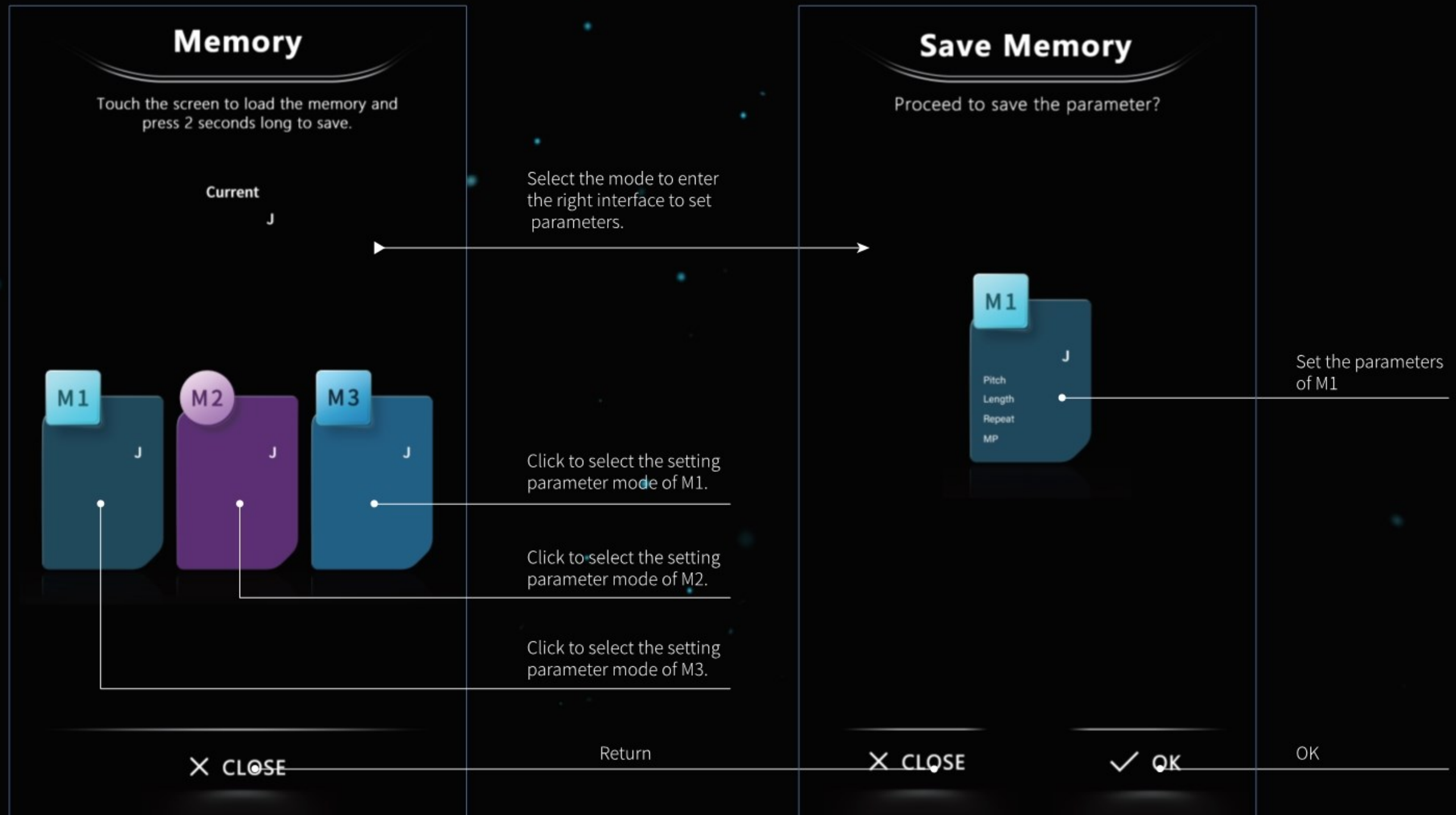

Operate the background Settings screen

	Adjust the sound of the operation.		The total number of strokes used by all models of the LEFT handle.
	It can be operated by handle buttons and foot pedals.		Clear all the LEFT handles.
	Play animated video		The total number of strokes used for each model of the LEFT handle.
	Instrument software version number.		The total number of strokes used by all models of the RIGHT handle.
	Factory date of the instrument.		Clear all the RIGHT handles.
	System background password settings.		The total number of strokes used for each model of the RIGHT handle.
	Restore initial settings.		The cumulative total number of rounds used for all handle models of the instrument.
	Return		




CONFIGURATION PARAMETERS AND MAINTENANCE MATTERS

- Instrument maintenance
- Instrument configuration parameter
- Safety tips
- Note instructions



Instrument maintenance

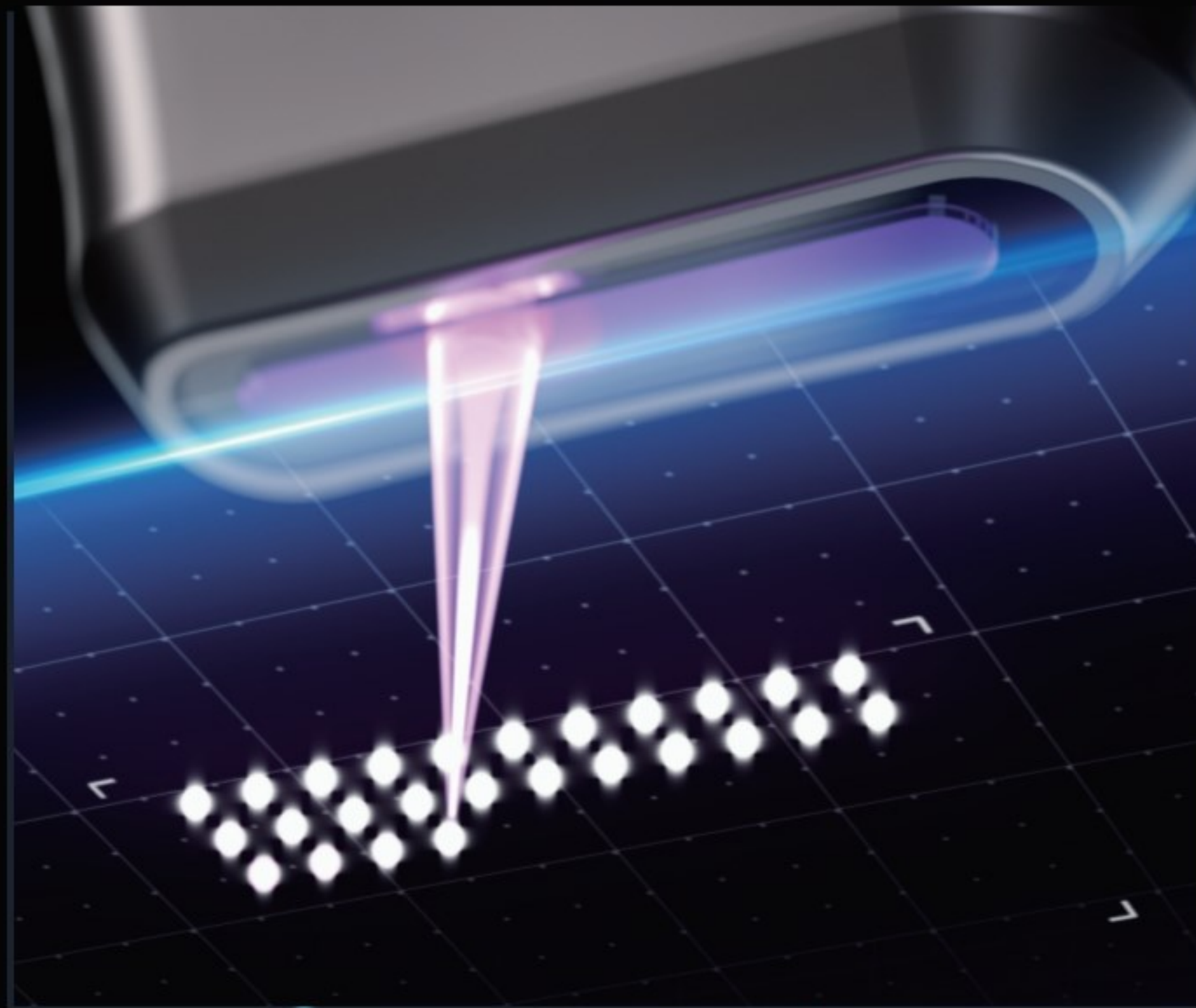

Maintenance precautions

1. The instrument is a precision device. Before and after use, please pay attention to cleaning and maintenance to avoid damage or shorten the service life. After use, please turn off the instrument switch in time, cut off the power supply and pull out the power cord.
2. The instrument should be handled with care. Do not pull the handle wire head and the host output wire head forcefully to avoid loosening the internal connection of the accessories, causing poor contact, short circuit or electric leakage.
3. After the instrument and probe are used, do not throw them away at will. Clean them carefully with cleaning solution and clean cloth. It is recommended to clean the surface with normal saline or a little detergent.
4. Ensure that the power supply voltage is consistent and stable with the power supply voltage specified by the instrument. If the local power supply voltage is unstable, it is recommended to increase the power matching regulated power supply.
5. In order to improve the care effect of the instrument and ensure its normal service life, please use the accessories provided by the original manufacturer uniformly, or use the designated accessories recommended by the original manufacturer.
6. The instrument should not be too close to strong heat source or strong radiation source (disinfection cabinet, induction cooker, microwave oven, etc.), which will affect the normal use of the instrument.
7. It is not allowed to hit the instrument or collide with force. In case of failure, it can only be repaired by personnel with professional maintenance qualification approved by the original manufacturer.
8. It is not allowed to disassemble the inside of the instrument alone. When liquid or something enters the instrument and cannot be taken out by itself, it can only be carried out by the personnel with professional maintenance qualification specified by the original manufacturer.
9. The main machine and accessories of the instrument should be cleaned and cared regularly, and special attention should be paid not to let the liquid splash into the instrument or enter into the instrument.
10. The instrument should not be placed in damp places or near water sources, nor should it be directly exposed to the sun.

Instrument configuration parameter

Product Name : 12D - MAX

Model : 12D - MAX

Power Supply : AC 110V/220V

Power : 1280W

Frequency : 2MHz 4MHz 5.5MHz

Instrument weight : 7 Kg

Gross product quantity : 20 Kg

Package size : 56.5X49.5X59cm



Instrument




Left handles



Right Handles



Cartridge

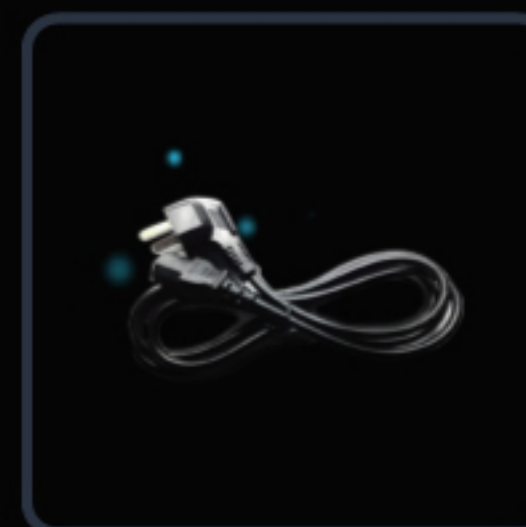

Power Cord

Safety tips

Professional tips:

Before use, please read the instructions carefully to use the instrument correctly.

This note is for the beautician to use this instrument safely and correctly, in case of emergency.

In order to indicate the size and extent of the hazard or damage, the incorrect use in this notice is divided into warning and caution.

This manual is specially marked with illustrations and icons. Its representative meanings are as follows:

In order for you to operate the instrument in a standard way:

- After understanding the meaning of the logo, please read this manual.
- After reading, please store this manual in a place where you can find it at any time.
- As the following contents are closely related to safety issues, please strictly comply with them.



Possible death or serious injury



Possible injury or damage to the instrument



The sign indicates that there will be danger, warning and caution, and the specific prompt content is recorded in the icon. (The left figure is applicable to no specific and general danger, warning and notice).



The mark indicates that there are prohibited behaviors, and the specific prompt contents are recorded in the icon and below the icon. (The left figure is applicable to no specific and general prohibition notices).



A mark indicates that there is an action that needs to be enforced or instructed, and the specific content of the enforcement or instruction is recorded in the icon. (The left figure is applicable to no specific general compulsion or instruction)

Note instructions



Pay attention

Precautions for use

- Understand and master the correct use of the instrument before operating the instrument.
- Cooperate with the company's designated skin care products for instrument care.
- Please take care step by step according to the conclusion of the customer's skin test and the degree of skin needs. When the customer's skin and physique reach the tolerance limit, please adjust the mode in time.
- Please confirm that the electrical specifications of all parts of the instrument are in good condition before use.
- In case of sudden power failure during operation, the power supply must be cut off in time to prevent the instrument from being burnt out by sudden power.
- After the nursing, make sure to cut off all power supplies related to the instrument.

Pay attention

- When installing the instrument, please be careful not to drop the instrument during loading and unloading.



12D MAX Instrument Manual

12D MAX